



URSATEC  
WHEN PRESERVATIVE FREE MATTERS®



SoftBreezer®

smart nebulizer  
breathe easily  
preservative-free

URSATEC dosage systems  
for preservative-free inhalation

# Preservative-free for a healthy life®

## The philosophy behind URSATEC

URSATEC is dedicated to support the global health care industry in avoiding preservatives in any kind of product. Health care administrations recommend the development of preservative-free products worldwide.

*„Devices and manufacturing processes shall be designed in such a way as to eliminate or to reduce as far as possible the risk of infection to patients, users and, where applicable, other persons.“*

Interinstitutional File: 2012/0266 (COD); Brussels, 27 June 2016 (OR. en); 10617/16  
Council of the European Union



## SoftBreezer® – innovative spray nozzle breathe easily

The SoftBreezer® spray nozzle unit is based on highly precise orifices on small surface area.



Pressing water through the SoftBreezer® spray nozzle unit results in multiple jets which will automatically break up in small droplets (Rayleigh principle). The resulting spray mist is mono-disperse, which means that all droplets have the same size for a good lung deposition.

# SoftBreezer® – refresh your life



## Refresh your life

First preservative-free mobile nebulizer for consumer health products such as isotonic saline inhalation solutions. SoftBreezer® is a manually actuated, pocket size nebulizer device for aqueous solutions, releasing a mono-disperse aerosol, targeting the airways very efficiently.

The SoftBreezer® preservative-free spray pump is based on URSATEC's proven spray devices, in use for more than 25 years.

## Key Features

Suitable for all kind of preventive products

- breathe easily
- Preservative-free
- Propellant-free
- Pocket size
- Proven 3K® technology
- Microbiologically safe

## Data sheet

**Pump delivery PD (water placebo)**

25 mg

**Airflow at 2 kPa**

10 -20 lpm

**Container fill**

5 ml or 10 ml

**Number of actuations/container**

200 or 400

**Microbial integrity of container in use**

min. 90 days



URSATEC Verpackung GmbH  
Marpinger Weg 4  
D-66636 Tholey

Telefon: +49 (0)6853 96199-0  
Telefax: +49 (0)6853 96199-14

E-Mail: [m-contact@ursatec.com](mailto:m-contact@ursatec.com)  
Internet: [www.ursatec.com](http://www.ursatec.com)